


**MESIRE™ Illuminus:**  
Light compatible light wire provides sinus cavity navigation and as an insertion aid for the balloon catheter. Illumination of the light wire allows precise placement.

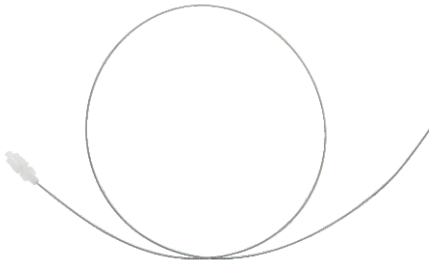

- Enables localization of ostium using transillumination flanking the ostium using an endoscope and Mesire Illuminus Sinus Light wire for confident confirmation of sinus access. The Mesire Illuminus Sinus Light Wire lights up the sinus it has entered with a diffused light visible on the skin in an area just somewhat larger than the sinus. The Endoscope lights up the other side of the ostium
- Distal-tip design enables responsive transmission of fine adjustment to seek visualized targets
- Compatible with light guide cable, connecting to a standard light source



Ordering Information:

Balloon Catheter	Sr. No.	Product Code	Balloon Diameter	Balloon Length
	1	MSB50017	5 mm	17 mm
	2	MSB60017	6 mm	17 mm
	3	MSB70017	7 mm	17 mm
Guide Catheter	Sr. No.	Product Code	Angle of Guide Catheter	
	1	MSG000	0°	
	2	MSG030	30°	
	3	MSG070	70°	
	4	MSG090	90°	
	5	MSG110	110°	
Guide Wire	Sr. No.	Product Code	Diameter of Light Wire	Lenght of Light Wire
	1	MSI083	0.83 mm	100 cm

www.merillife.com

**Meril Life Sciences Pvt. Ltd.**  
Survey No. 135/139, Bilakhia House,  
Muktanand Marg, Chala,  
Vapi 396 191.  
Gujarat. India.  
T +91 260 3052 100  
F +91 260 3052 125

Meril, inc.  
116 Village Blvd,  
Suite 200,  
Princeton, NJ - 08540  
T +1 609 951 2287  
F +1 609 520 1702

Meril GmbH.  
Bornheimer Strasse 135-137,  
D-53119 Bonn.  
Germany.  
T +49 228 7100 4000  
F +49 228 7100 4001

Meril South America  
Doc Med LTDA  
Al. dos Tupiniquins,  
1079 - Cep : 04077-003 - Moema  
Sao Paulo. Brazil.  
T +55 11 3624 5935  
F +55 11 3624 5936

Meril Tibbi Cihazlar  
Imalat Ve Ticaret A. S.  
6, Mimar sinan Mah.,  
Çavusbasi Cad. Özde Sok.  
Aydin Eksi Is Merkezi Kat:1  
Cekmekoy/Istanbul, Turkey  
T +90 53 2272 5172

**E** askinfo@merillife.com  
**W** www.merillife.com

EU Representative  
Obelis S. A.  
Bd, General Wahis 53,  
1030, Brussels, Belgium  
T +32 2 732 5954  
F +32 2 732 6003  
E mail@obelis.net

Meril Brazil  
Alameda Maracatins,  
992 Bloco B 8 Andar  
Indianópolis - Sao Paulo,  
SP Brazil. Zip Code: 04089-001

www.merillife.com



# Meril Sinus Relief



**MESIRE™**  
Balloon Sinus Dilation Products




MESIRE™

Balloon Sinus Dilation Products

Sinusitis Overview

Sinusitis is defined as an inflammation of the sinus lining, and is commonly caused by structural issues such as ostial blockage, bacterial infections, viral infections or a combination of these. For many patients, symptoms vary and may include nasal congestion, facial discomfort, nasal discharge, headache and fatigue. Sinusitis is considered acute when symptoms last 4 weeks or less and chronic when it lasts 12 weeks or longer.

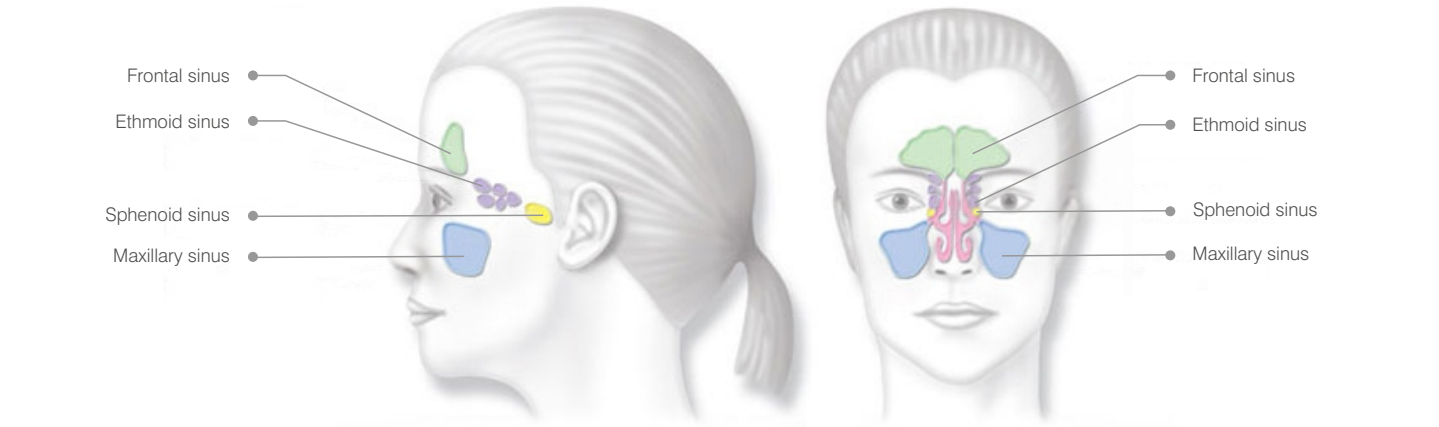

Balloon Sinus Dilation is safe and effective procedure for chronic sinusitis patients who are not responding well to medications and seeking relief from uncomfortable and painful sinusitis symptoms.

In Balloon Sinus Dilation ENT surgeons open inflamed sinuses in the same way that cardiologists open up blocked arteries during balloon angioplasty. Mesire Products used the teachings of Mark A Saab in 'Applications of High-Pressure Balloons in the Medical Device Industry' published September 2000 in Medical Device & Diagnostic Industry Magazine. Mesire Products also use the direct illumination and transillumination from placement of light sources in the sinus cavities reported in US Patent 7,520,876.

Balloon Sinus Dilation is less invasive than traditional sinus surgery and effective at relieving symptoms of chronic sinusitis. Meril Balloon Sinus Dilation allows patients to return to normal activities quickly unlike conventional sinus surgery. Meril sinus surgery does not include removal of bones or tissue from the nose.

<div>Safe:</div> <div><div>- Safe design (eg. atraumatic tip)</div><div>- Zero adverse events</div><div>- Less general anesthesia time and necessity</div></div>	<div>Effective:</div> <div><div>- Patency: Extremely low patency &amp; revision rates</div><div>- CT Scan: Consistent improvement in Lund-MacKay CT scores</div><div>- Symptoms: Significantly improved SNOT-20 score</div><div>Symptoms/QOL scores</div></div>
<div>Patient Friendly:</div> <div><div>- Less invasive</div><div>- Less trauma</div><div>- Less pain</div><div>- Less recovery time</div><div>- Less scarring</div><div>- Less follow-up procedures</div></div>	<div>Enabling:</div> <div><div>- Find and treat the frontal sinus/recess</div><div>- Help special case patients (eg. bleeders, ICU)</div><div>- May enable paradigm shift to local anaesthesia</div></div>

In Balloon Sinus Dilation, inflamed sinuses are opened. After Balloon Sinus Dilation the sinuses remain open, allowing the return of normal sinus drainage and relief of sinus pressure.

Safe & Effective

**Less Invasive** – relieves pain and pressure of chronic sinusitis without cutting or removal of natural structures in the nose.

**Fast recovery** – many people return to normal activities within 24 hours.

Proven

**Reliable** – 350,000+ have benefited from Balloon Sinus Dilation

**Mandated** – procedure approved by India DGHS & European CE work.

INDICATIONS:

Balloon Sinus Dilation Products

1. Chronic Rhinosinusits (CRS)

Patients with history of persistent rhinosinusitis (all sinuses) lasting greater than 12 weeks, despite medical management of antibiotics, topical steroids and allergic management or patients with persistently abnormal CT after at least 4 weeks of continuous antibiotic treatment.

2. Recurrent Acute Rhinosinusitis

An abnormal CT during treatment with post-treatment normalization with 3 or more recurrences per year with each episode lasting less than 2 weeks.

3. Revision Surgery

Patients who have undergone ESS (once before or multiple times) with persistent symptoms and opacification on CT scan with a narrow Ostia. Balloon Sinus Dilation is a safe and effective tool in difficult revision surgeries with distorted post-operative anatomy.

Why Balloon Sinus Dilation?

The Balloon Sinus Dilation is an advanced surgical procedure approved in India by Indian DCGI and European CE mark approved product. Balloon sinus dilation uses a precise single use balloon catheter to open inflamed sinuses. The balloon catheter is placed into the sinus to reach the inflamed sinuses and gently inflated to restructure the previously blocked nasal passage.

The Balloon Sinus Dilation preserves the normal anatomy of the sinuses and mucosal tissue, and does not require the removal of bone and tissue – this approach helps restore normal sinus functioning. FESS involves removal of bone and tissue and the recurrence rates are high.

Since the disease state is minimal, it's not required to perform extensive procedure as no major abnormalities are associated with the disease.

The patient has understood the pros and cons of both the procedure and has opted for sinuplasty which has been the standard of care in many countries today.

Benefits of Balloon Sinus Dilation Procedure:

- Less invasive procedure


- Lesser blood loss

- Associated with fewer recurrences

- Post operative recovery is faster and patient can resume work quickly

- Safe and effective minimally invasive procedure for the treatment of chronic sinusitis


Balloon Sinus Dilation Procedure:

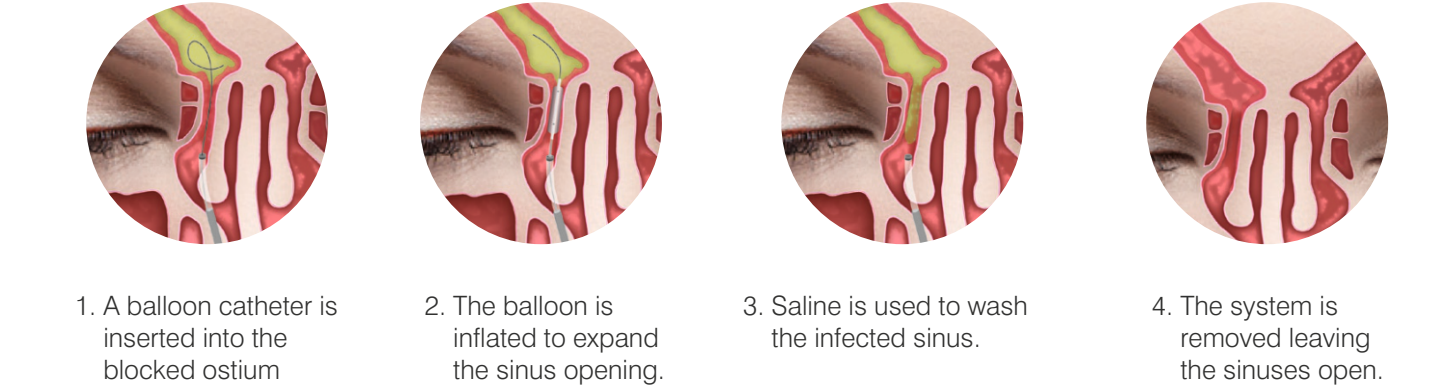

Product Overview:

MESIRE™ Balloon Catheter:

MESIRE™ Balloon catheter is non-compliant balloon catheters designed for precise control on diameter of balloon expansion. It enables accurate placement of balloon in the sinus ostia through visual indicators.

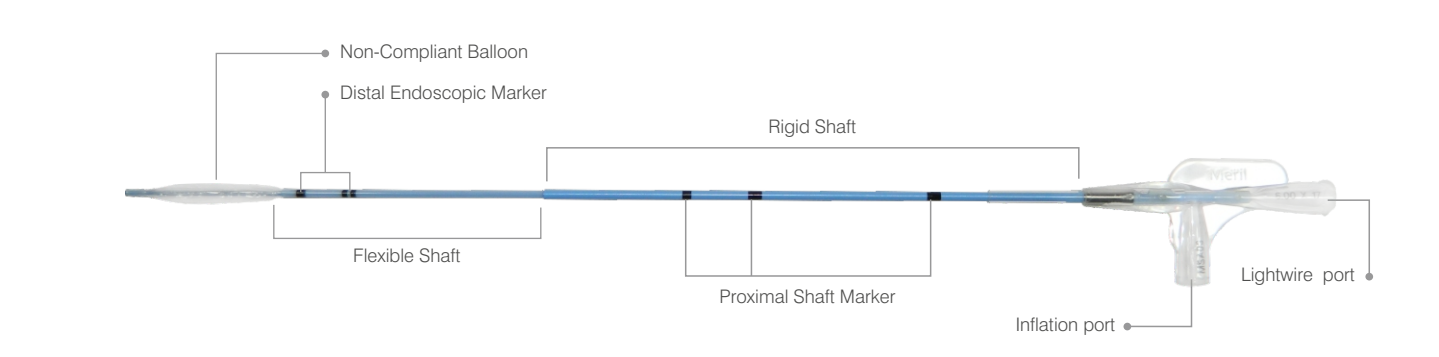

Precise balloon placement

- Endoscopic markers provide information on balloon location under direct visualization

- Enhanced shaft coloring and markers provide improved visibility under low light

Enhanced tracking and retraction

- Optimized balloon tapers

- Six-fold balloon design

- New balloon insertion stylet enabling smooth balloon insertion into the sinus guide catheter

Increased procedural efficiency

- Precise re-wrapping of balloon

- Less balloon deflation time\*

MESIRE™ Guide:

Mesire™ guide catheter with a distal angle of 0°, 30°, 70°, 90°, 110° are designed for targeted navigation of the Mesire™ balloon catheter into respective ostium. Mesire™ guide is having soft atraumatic tip which prevents the tissue trauma during the insertion.

